

F&P SleepStyle Non-modem Data Transfer Options



As an alternative to using the inbuilt 3G cellular modem, data uploading to InfoSmart™ Web can also occur via the SleepStyle™ App or via InfoSmart™ Web at their next visit.

Uploading SleepStyle therapy data via the SleepStyle App



The SleepStyle App uses Bluetooth™ wireless technology to communicate with the F&P SleepStyle device and automatically upload sleep data from the device to the InfoSmart Web for you to access.


To upload data from the F&P SleepStyle device via Bluetooth onto the App (which is then automatically uploaded onto the InfoSmart Web cloud), all the patient needs to do is ensure:

- Their F&P SleepStyle device is plugged into a power supply and switched on
- Their mobile phone's Bluetooth function is switched ON and previously paired with their F&P SleepStyle device
- Their SleepStyle App is open and running on their smartphone to receive and transmit data.

Please note:

- Up to 12 months of therapy data can be transferred.
- Prescriptions cannot be sent via the SleepStyle App to the F&P SleepStyle device.
- If your patient needs to download the SleepStyle app on their phone, please refer to the step-by-step "SleepStyle App Pairing guide UI-617114_A" document attached. They can download the SleepStyle App via The App Store or Google Play™ for Android. The patient can also refer to section "4.2 View Your Therapy Data on the SleepStyle App or Website" of the F&P SleepStyle Use & Care manual for detailed instructions on how to pair your F&P SleepStyle device to your mobile device.

Uploading SleepStyle therapy data via InfoSmart Web during a store or clinic visit

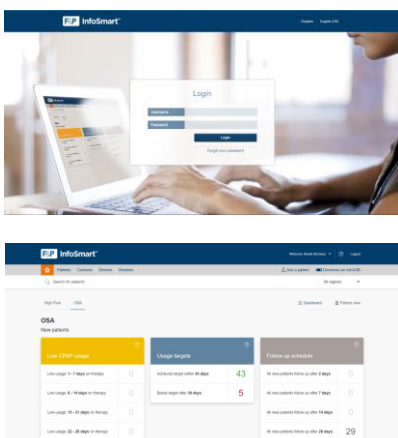

You are able to upload your patient's sleep data to the InfoSmart Web Cloud via your computer, during their next appointment.

All you need to do is:

- Ask your patient to bring their F&P InfoUSB to the appointment
- When the patient arrives for their appointment, retrieve the F&P InfoUSB from them and insert it into your computer
- Log into your InfoSmart Web Account at: <https://www.fpinfosmart.com.au/>
- Once in, click on the "Download an InfoUSB" button in the top right hand corner and follow the prompts to upload the data from the InfoUSB to the patient's account in InfoSmart Web. Depending on the amount of data to be transferred, this upload could take a few minutes to complete
- Once the data is uploaded, return the InfoUSB back to the patient and remind them to return it to their SleepStyle device